

The Pivotell logo, featuring the brand name in a blue, stylized font enclosed within a thin blue circular border.

Pivotell



Vibratime

12 Alarm vibrating watch

For instructions in German, French, Italian, Spanish or Dutch, please visit:

<https://www.pivotell.co.uk/Pivotell-Downloads.htm>

DE *Anweisungen in Deutsch, finden Sie unter dem obigen Link.*

FR *Pour des instructions en Français, veuillez visiter le lien ci-dessus.*

IT *Per istruzioni in Italiano, visitare il link sopra.*

ES *Para obtener instrucciones en Español visite el enlace de arriba.*

NL *Voor instructies in het Nederlands, gaat u naar de bovenstaande link.*

Introduction

1

A possible **12 silent vibrating alarms** can be set as convenient & discrete reminders to help adults and children remember to do something during the day or night.

There are two programming buttons which can be felt on the side of the strap.

To check the time: Press the **right** button once. The time will appear on the screen.

To check the battery level: Press the **right** button twice. The battery level percentage will appear. (If the battery is below 20% the watch will turn off to save the alarm settings).

To review all alarms: repeatedly press the **left** button. The alarm number is followed by the alarm time. LED will show: **A1OF**, 08.08, **A2OF**, 09.00, **A3OF**, 10.00 & so on, for all 12 alarms.

At an alarm time the watch vibrates and time is displayed. Alarm 1 vibrates for 40 seconds (wake-up alarm). Alarms 2-12 vibrate for 20 seconds.

When the watch alarm vibrates you can press either button to stop the alert.

When setting the watch, press and hold the appropriate button for 3 seconds to enter the setting. A short press adjusts the number.

Practice with the settings initially as it takes a little while to get used to them. The LED turns off quickly (exiting the setting) if you are not prompt when pressing the buttons.

If nothing is pressed the watch will exit setting after 4 seconds.

Watches are supplied with factory set demo alarm times. Adjust the alarms to suit your needs.

Setting the Time

3

1, Press the **right** button once - the 24 hour clock will appear.

2, With the LED still on press the **right** button again and hold for 3 seconds to set the hours.

4, The hours will appear alone on the left (central dots will be blinking).

5, With the LED still on press the press the left button to increase, the right button to decrease.

6, When the hour is correct and with the LED still on press the **right** button for 3 seconds to set the minutes.

7, The minutes will appear alone on the right (central dots will be blinking).

8, With the LED still on press the left button to increase, the right button to decrease.

When the minutes are correct - time setting is complete.

If nothing is pressed the watch will exit setting after 4 seconds.



Setting the Alarms 5

1, Turn alarms on/off.

Alarm 1: Press the **left** button once. With A1OF (Alarm 1 OFF) or A1ON (Alarm 1 ON) displayed, press the **right** button once to turn **ON** or **OFF**.

2, Set the alarm time.

Alarm 1: With A1ON displayed press the **left** button once to see the alarm time on the LCD.

A, Then press the **left** button for 3 seconds to enter the setting and adjust the hours. The hours will appear on the left side.

B, Press the left button to increase, the right button to decrease.

C, Once at the correct hour, press the **left** button for 3 seconds to adjust the minutes. The minutes will appear on the right side.

D, Press the left button to increase, the right button to decrease. **When the minutes are correct, alarm 1 is set.**

Now program any further alarms required (2-12):

To turn an alarm on/off ensure the correct alarm is displayed by pressing the **left** button until it is displayed (ie. A2OF, A3OF, etc.) and press the **right** button once to change it to ON or OFF.

Set the times for alarms 2-10: With the correct alarm displayed (ie. A2ON, A3ON, etc.) press the **left** button once to see that alarm time on the LED. Then follow instructions **ABCD** above.

Recharging

7

Remove the watch from the rubber casing and charge the watch using the USB cable provided.

The watch charges in approximately 1 hour.

When replacing the watch in the rubber casing ensure that the watch buttons are correctly positioned in the strap under the rubber buttons.

Battery life

Battery life between charging: appx 6 days, although this will depend on the number of alarms set & length of time the vibrations are allowed to run / frequency LED light is on. At 20% or less the LED will display __:00 after the time as a reminder to recharge.

We recommend that the Vibratime is not recharged every day unless necessary.

The Vibratime is not waterproof: do not wear in shower/bath or when swimming.

Warranty: 12 months from purchase.

Other interchangeable strap colours also available:



Also in our range



www.pivotell.co.uk